


 **Please Read Carefully Before
Applying ProCollar to your pet**

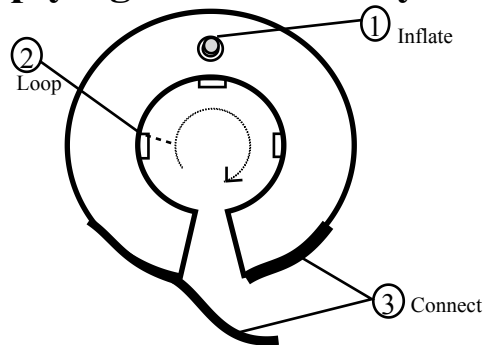

(1) Inflate ProCollar via air valve. (Do not use a compressor)

(2) **To Secure ProCollar in place:**

- a. Loop your pet's *everyday collar* through the three inner rings of the ProCollar.
- b. Securely fasten pet's everyday collar.

(3) Connect Velcro strap on the ProCollar.

- ✓ *For larger breeds with longer limbs, or breeds in between sizes, always size up!*
- ✓ *Always monitor a pet post-surgery to ensure collar is working properly. Some pets may be able to maneuver around any collar.*